F

PROPRIÉTÉS

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UK

PROPERTIES

The DETARTRINE range is available as ready-to-use pastes designed for dental scaling and polishing.

- **DETARTRINE** and **DETARTRINE Z** are pink pastes, with a nice taste. Their mechanical abrasive action is due to the presence of silica, which makes it possible to complete the removal of dental tartar, after using an ultrasonic or hand scaler, without

causing erosion of dental enamel.

DETARTRINE Z contains zirconium particles. These particles look like flat scrapers, the sides of which are separated by a very hard edge. When this paste is rubbed against a rough surface, the particle edges shave the bumps off. The essential oils incorporated into DETARTRINE and DETARTRINE Z give a feeling of freshness, which is very much appreciated by the patients. In addition, they have a slight balsamic effect: they relieve the irritation that mechanical scaling might cause on sensitive gums.

- **DETARTRINE WITH FLUORIDE** is a white paste with a nice flavour, formulated with zirconium oxide and sodium fluoride and designed for dental polishing. Due to its fine and hard particles, zirconium oxide quickly gives a gleam to enamel surfaces, metal and acrylic restorations and dental cements. It also complements chemical and mechanical scaling, giving the tooth surface a beautiful finish and reducing the adherence of dental plaque. Sodium fluoride was included in the formulation to promote topical fluoridation of newly exposed enamel, in order to protect it from possible future chemical attacks.

INDICATIONS

DETARTRINE – DETARTRINE Z

- Finishing of scaled surfaces,
- Removal of food and tobacco stains,
- Fine polishing of fillings.

DETARTRINE WITH FLUORIDE

- Polishing after scaling,
- Polishing of fillings.

INSTRUCTIONS FOR USE

Take the required amount of paste with a spatula and place it onto a polishing wheel. DETARTRINE WITH FLUORIDE can also be applied using little cups.

Note: Mouth rinsing can be performed during or at the end of scaling.

PRECAUTIONS FOR USE

Possible allergic reaction to formaldehyde.

PRESENTATIONS

45 g jar.

150 g jar.

For professional use only in the practice of dentistry.

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