Gelopack

España de gelatina hemostática, estéril, reabsorbible

Propiedades

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Sterile, resorbable, haemostatic gelatine sponge

Properties

The GELOPACK gelatin sponge is made of 100% purified porcine gelatin. It is intended for the use as an absorbable haemostat in dental surgery. It is insoluble in water and fully resorbable. Its porous structure and interstices enables the product to absorb more than 40 times its own weight of blood and fluids.

It is packaged individually in blister.

Indications

General indications: the GELOPACK gelatin sponge is used during and after dental surgical procedures to achieve local haemostasis, especially for example where conventional procedures are either ineffective or impractical. In Dental surgery, the GELOPACK gelatine sponge is an aid in providing haemostasis and obliterating dead space created by extraction of the teeth, root amputations and removal of cysts, tumours and impacted teeth.

Contraindications

Allergies to porcine products.

The product should not be used without antibiotics in infected wounds.

Side effects

There have been no reported adverse effects for product used correctly according to these Instructions For Use and when over-packing is not applied.

Directions for use

Pull out the required number of sponges with a sterile instrument.

GELOPACK can be used dry or after being moistened with a physiological solution.

- If applied dry, GELOPACK sponge should be compressed before application to the bleeding surface and held in place with moderate pressure for a short time.

- If used moistened, GELOPACK sponge should be squeezed thoroughly before application.

When bleeding is controlled, the sponge can be left in the socket after removing a possible excess of material. The sponge is completely resorbed after a few weeks.

Warnings and precautions for use

In cases where the gelatin sponge has been compressed, the gelatin sponge may expand with the absorption of fluids (the sponge will obtain its original shape). Where possible GELOPACK should be removed after usage and bleeding has stopped.

The over-packing of GELOPACK, particularly within bony cavities, should be avoided, since recovering to its initial volume may interfere with normal function and/or could cause possible or eventual compression necrosis of surrounding tissue.

Formation of tissue granulation has been observed during ENT / middle ear procedures.

In cases of postoperative infection, re-operation may be necessary to remove infected material and allow drainage.

Do not use when the package has been opened or has been damaged.

The absorbable gelatin GELOPACK sponge is a single-use product which is not suitable for re-sterilization.

Storage

The product should be stored in a dry place  in its protective packaging at room temperatures .

Avoid contact with direct sunlight .

Prior to opening, the packaging ensuring the integrity of the sterile barrier has to be inspected. Open the package just prior to use in a patient.

Presentation

Box containing 3 blisters of 10 sponges (10 x 10 x 10 mm).

Sterilised by gamma irradiation.

For professional use only in the practice of dentistry.

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