Résorcell,

bolitas hemostáticas reabsorbibles

Composición

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resorbable haemostatic pellets

Composition

Oxidized cellulose (contains 16 to 24 percent of carboxyl groups calculated on dried substance basis).

Indications

Haemostasis of capillaries and small vessels after tooth avulsion.

Instructions for use

Apply a small piece of RESORCELL just enough to cover the wound, then place a sterile compress over it and ask the patient to keep his/her teeth clenched so as to exert a firm but not exaggerated pressure for 2 or 3 minutes, then remove the compress.

When the wounds are poorly irrigated or when the osseous tissue is exposed, it is better to use the following method as it will permit the formation of a physiological clot when in contact with the vital tissues : introduce a cube of haemostatic fibrin sponge at the base of the alveolus so as to cover the exposed osseous tissue. Then insert several small pieces of RESORCELL to fill up the alveolus completely. Ask the patient to bite up till bleeding stops. Afterwards remove any excess RESORCELL.

Warnings and precautions for use

RESORCELL, should not be applied directly to osseous tissue when irrigation with blood is not sufficient because the acidity of the product causes callus formation slowing down.

Avoid stitching tissues after insertion of RESORCELL.

Do not leave RESORCELL in contact with bone in a closed cavity. The oxidized cellulose may provoke an important pressure by swelling according to the quantity placed in situ and the irrigation of tissues with blood.

RESORCELL is not designed for superficial wound as it slows down epithelization.

RESORCELL is not compatible with local application of penicillin.

Filling in excess the wound with the product is unfavourable to haemostasis, as this will result in a delayed cellulose resorption.

RESORCELL should be used according to suitable procedures so as to reduce the risks of microbial contamination.

RESORCELL should not be impregnated by anti-infectious substances or buffered solutions.

The use of RESORCELL simultaneously with thrombin is inappropriate.

Storage

Keep the product in a cool place and away from moisture. Tightly close after use.

Presentations

Bottle containing 50 pellets.

For professional use only in the practice of dentistry.