FR

**COMPOSITION**

Chlorure d'aluminium 25 % (m/m), excipients.

**PROPRIÉTÉS**

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25 % aluminium chloride (w/w), excipients

**PROPERTIES**

This gel helps to prepare the sulcus prior to impression taking and can be used with or without gingival retraction cord.

Its thermogelifiable formula, combined with its astringent properties, facilitates the opening of the gingival crevice and reduces bleeding and oozing.

Thanks to its consistency, it is easily removed by rinsing. This gel allows rapid and optimal preparation of the gum tissue before impression taking and sub-gingival restorations.

**INDICATIONS**

- Sulcus preparation prior to impression taking and sub-gingival restorations.

- Control of bleeding and gingival oozing, particularly in restorative dentistry.

- If using a gingival retraction cord, the gel facilitates the insertion of the cord into the sulcus.

**CONTRA INDICATIONS**

Allergy to aluminium, aluminium salts and excipients.

**SIDES EFFETS**

Allergic reaction, burning sensation.

**INSTRUCTIONS FOR USE**

- Remove the cap from the syringe.

- Fix the pre-bent tip.

- Apply a sufficient amount of gel directly into the sulcus, carefully following the contour of the prepared tooth.

- Depending on the clinical situation, you may pack a retraction cord into the gingival crevice.

- Leave in place for about 2 minutes.

- If bleeding persists, rinse, reapply gel and wait again about 2 minutes.

- Rinse.

- Gently dry with an air syringe.

- Take the impression.

**WARNINGS AND PRECAUTIONS FOR USE**

Warning

Causes skin irritation.

Causes serious eye irritation.

If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

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