COMPOSITION

Eugenol, essence de menthe badiane contenant (E)-anethole et dipentene.

PROPRIÉTÉ

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EN

COMPOSITION

Eugenol, Peppermint oil containing (E)-anethole and dipentene.

PROPERTIES

Component needed to obtain the eugenate cement.

PERFORMANCES

Eugenol of Endomethasone Liquide reacts with the Zinc oxide of Endomethasone powder range to form eugenate cement.

INDICATIONS

Liquid to be mixed with Endomethasone powder range to prepare permanent root canal filling cement.

CONTRA INDICATIONS

Hypersensitivity to one of the components.

ADVERSE EVENTS

Risk of allergy to any of the constituents particularly in case of contact with gums.

INSTRUCTIONS FOR USE

Prepare the root canal filling paste extemporaneously following the instructions for use of the concerned Endomethasone Powder.

Close vial tightly after use.

Fill the root canals using a paste-carrier and insert into the paste one or several points after adapting it/them to the canal shape.

Take X-rays to ensure that the canals have been thoroughly filled.

INTERACTIONS

Do not use in combination with composite materials since eugenol-based preparations inhibit their polymerization

WARNINGS AND PRECAUTIONS FOR USE

The canal can only be sealed if it has previously been prepared, cleaned and dried.

Determine the adequate working length of the root canal before filling so as to prevent any apical extrusion, and appropriate sealing.

Warning

Contains Eugenol, (E)-anethole, dipentene.

Cause serious eye irritation.

In case of contact with eyes, immediately rinse the eye with plenty of fresh water for several minutes and contact a doctor.

May cause an allergic skin reaction.

It must not be diluted.

Harmful to aquatic life with long lasting effects.

Refer to safety data sheet (available upon request).

STORAGE

Do not store above 25°C.

PRESENTATION

10 ml bottle

For information:

The content of the 10mL bottle is estimated to ensure a maximum of 110 uses.

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