COMPOSITION

Chlorure d'aluminium hexahydraté 25 % m/V, oxyquinol, excipient hydroalcoolique.

PROPRIÉTÉS

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(EN)

COMPOSITION
Hexahydrated aluminium chloride 25 % w/V, oxyquinol, hydroalcoholic excipient.

PROPERTIES

Aluminium chloride has an astringent effect. It induces local haemostasis and the contraction of the upper strata of the free gingiva by precipitating tissular and serum proteins.

INDICATION

- gingival exudation and gingival haemorrhages,
- gingival retraction by *in situ* impregnation of a cord.

CONTRAINDICATIONS

Possible allergy to aluminium.
Any history of hypersensitivity or photosensitivity to quinoleic ingredients.

SIDE EFFECTS

Allergic reaction, burning sensation.

INSTRUCTIONS FOR USE

- Gingival exudation and gingival haemorrhage** Instil one or two drops of solution onto the gingiva using a pipette or a cotton swab. Wait 1 to 2 minutes and thoroughly rinse the application site.
- Gingival retraction** First insert a (non-medicated) gingival retraction cord into the sulcus. Carefully impregnate the cord *in situ* with a few drops of solution, using a pipette. Leave the cord in place for 3 to 5 minutes. Remove it, making sure that no fragment remains into the gingival sulcus. Wash thoroughly.

WARNINGS AND PRECAUTIONS FOR USE

- Instil only the number of drops required to obtain the desired result.
- Reduce to a minimum the contact time of the solution with the mucosa.
- As soon as local haemostasis is obtained or the impression performed, thoroughly wash the application site so that there is no potentially irritating aluminium salt left.
- Not to be used for over-impregnation of a medicated cord.
- Do not swallow.
- Warning Flammable liquid and vapour. Cause skin irritation. Cause serious eye irritation. Refer to safety data sheet.

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