(FR)

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(EN)

Instruction For Use

Easier Filing with Lubricating Action

Applications

Root canal preparation with files and reamers

Presence of calcification in the canal

Too narrow canals in posterior teeth

Smear layer removal

Mode of action (Mechanism Principle)

Removal of inorganic substances by chelation (EDTA - Ca)

Easier filling due its lubricating action

Produces the cleanest canal wall after smear layer removal

Composents

Canal + Cream x 7,0g (5,4 ml) x 2

- Disposable tips
- Net weight : 14g

Directions for use

- Deposit a small amount of Canal + Cream in the canal orifice
- Carefully pump into the canal with a fine file or reamer
- Re-apply Canal + Cream when necessary or until cleaning and shaping is complete

How to use Rotary Ring

- If in contact with air, content could be hardened, so it is necessary to cover a cap on the needle after use

- Putting a syringe into a rotary ring is more convenient to handle & Use the rotary ring as a wrench to tighten or loosen a tip

Cautions

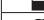
- Use Canal + Cream with discretion to prevent perforation
- Avoid extrusion of the solution into the apex
- Keep the syringe capped all the time

Storage

Store at room temperature 1°C–30°C (33°F–86°F)

Shelf life

2 years from the manufactured date.

	Manufacturer
	Date of manufacture
	Batch Code
	Use By
	Consult operating instructions
	CE - Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC
	Keep away from sunlight
	Temperature limitation
	Authorized representative in the EC

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